

The Clubfoot Solutions App: Instructions for Customizable Brace Wearing Schedule

The schedule for wearing a clubfoot brace is completely customizable on the Clubfoot Solutions App. You may enter the Name of the Brace and the Brace Start Date at the top of the screen as seen below. The Custom toggle will be selected. This will be found directly under the Brace Start Date. Continuing down the screen, the Start and End Date is entered along with the prescribed hours by your child's doctor to the left. This information must be saved by hitting the Save button at the bottom of the display. If information is incorrect, it may be canceled, and user or health care provider may reenter data.

The screenshot displays the 'Profile' screen of the Clubfoot Solutions App. At the top, there is a green header with 'Profile' and 'UIOWA SURVEY' with a back arrow. Below the header, the text 'Iowa Brace' is entered in a text field. Underneath, 'Brace Start Date' is set to '05/12/2016'. A 'Custom' toggle is turned on, with a red 'Cancel' button next to it. Below this, there are four rows of schedule entries, each with 'Hours', 'Start date', and 'End date' fields, and a checkmark icon to the right. The entries are: 11 hours from 04/12/2018 to 04/18/2018; 8 hours from 04/19/2018 to 04/30/2018; 6 hours from 05/01/2018 to 11/30/2018; and 09/06/2018 to 09/06/2018. At the bottom, a numeric keypad is visible with a blue checkmark button.

Hours	Start date	End date	Action
11	04/12/2018	04/18/2018	Save
8	04/19/2018	04/30/2018	Save
6	05/01/2018	11/30/2018	Save
	09/06/2018	09/06/2018	Save